



The nations #1 manufacturer of hospital cribs
CLEANING INSTRUCTIONS
FOR HARD MANUFACTURING
CRIBS, BEDS & MATTRESSES

CRIBS OR BEDS

- ◆ HARD Manufacturing recommends the cribs and beds be washed by hand with a non-abrasive cleaner of your facility's choice and then thoroughly dried with a soft cloth or towel. Our equipment should never be placed in a machine for cleaning or subject to steam guns or pressure washers of any kind. Do not allow the crib or bed to air dry as pools of water left on the mattress pan surface or near bolt holes may eventually lead to premature corrosion.
- ◆ We suggest the use of a cleaner such as one that contains Quaternary Ammonium Chloride (Quats). Quats will both clean the crib or bed as well as disinfect it without oxidizing our epoxy.
- ◆ Any type of cleaner containing ammonia, alcohol, bleach or any other chemical that is abrasive should never be used on our cribs or beds for any reason.

MATTRESSES

- ◆ For mattresses, HARD Manufacturing recommends they be wiped down with a cleaner/disinfectant of the facility's choice. Our mattresses should not be subjected to any type of power washing. Bleach should never be used on our mattress as it may discolor the fabric.
- ◆ Mattresses should never be put back on the crib if the cover is not thoroughly dry. Failure to do so may cause premature corrosion on the pan mattress surface.
- ◆ Mattresses should always be allowed to dry in a flat position, never in an upright position. Failure to keep the mattress flat during the drying process could lead to the foam inside the cover to shift, creating permanent damage/lumps in the mattress.

Please note failure to follow the recommended cleaning guidelines stated by HARD Manufacturing can cause permanent damage to the crib/bed and/or mattress which will not be covered under warranty.

HARD Manufacturing recommends our equipment be cleaned at minimum once per year, or as often as required/needed per end user procedures/guidelines.

